



Figure 1: The Michelangelo Hand © 2014 Ottobock

## Getting a second grip on life

Losing a hand in a tragic event turns everyday tasks into a constant battle. Nothing is the way it was before and even small things suddenly seem impossible. The high-tech Michelangelo Hand prosthesis gives back quality of life to people who lose a hand. maxon DC brushless motors play an important role.

The human hand is a masterpiece of nature and helped us to become what we are today. We use our hands to assemble the tiniest clockworks, to throw balls, or to communicate when language barriers become difficult. This makes life very hard for people who have lost a hand due to an injury or an accident. Patrick Mayrhofer personally understands such struggles. In a job related accident, he injured his hands so badly that he chose amputation of his left arm shortly afterwards. Yet, the young man did not let this fate knock him down. "I'm very focused on my goals, and when I put my mind on something, I usually succeed at it." He does not want any special treatment and he does not need it either. With his new prosthetic hand, he masters practically all day-to-day situations without difficulties.

### **The impossible becomes possible**

His prosthesis, called the Michelangelo Hand, was developed by Ottobock in Vienna. It can perform seven different gripping movements, which the user can trigger by contracting the muscles in his stump. Tasks that were previously impossible suddenly become possible: Balancing a plate on one's palm, taking an egg from a box, holding a menu, or even peeling a banana. In the words of Martin Wehrle, another Michelangelo user: "With many movements, I simply grab the object without thinking much about it."

Stefan Roschi © 2014 maxon motor ag

**maxon motor**

driven by precision

To replicate the human hand as closely as possible, the engineers at Ottobock needed to experiment for many years and had to miniaturize all the electronic and mechanical components in the prosthesis. As a result, it weighs only 520 g and is comfortable to wear. The prosthesis does not only look natural, but it also feels natural - for example when shaking hands. This is due in part to the artificial wrist with its dampened movements. The mechanical systems enabling the gripping functions are driven by a brushless maxon motor, type EC 10, and an adapted version of the EC45. The first motor drives the thumb, the second is in charge of the main drive for the index and middle fingers. The ring finger and little finger move passively. The EC motor for the thumb has an ironless winding and is also equipped with a worm shaft customized by maxon motor.



*Figure 2: With the Michelangelo prosthesis, holding bowls or plates is made easy. © 2014 Ottobock*

### **Drives with high power density**

maxon motor has been supporting Ottobock with special solutions since the company began its development of Michelangelo. Requirements to the drive system include a high power density, smooth running, and resistance to high axial shaft loads. maxon brushless DC motors meet all these requirements. Additionally, they have a long life span.

### **Sensor for reliable gripping**

The interface with the human body is another important aspect of the Michelangelo prosthesis. Ottobock uses electrodes to measure the electrical pulses in the muscles at the stump and sends these signals to a processor. The Axon-Bus data transfer system is extremely fast and reliable. It makes handling the prosthesis easy and intuitive for the user. The harder the user tenses their muscles, the faster and stronger the hand grips. Simultaneously, a sensor in the thumb measures the closing force. This allows for accurate control of the gripping force when an object threatens to fall down.

The Michelangelo Hand gives many users the chance of returning to work. This is especially true of people in the prime of their life who have a family and want to work. These people can profit from the many applications of this prosthesis. Hans Dietl, managing director of Ottobock, says: "We want to give people the highest possible degree of mobility and independence. We work on this daily and will continue to do so in the future."



Figure 3: A maxon EC 10 motor is used in the Michelangelo prosthesis. The brushless DC drive provides a power of 8 W.  
© 2014 maxon motor ag

Author: Roschi Stefan  
3779 characters, 769 words, 3 figures

For additional information, contact:

maxon motor ag  
Brünigstrasse 220  
Postfach 263  
CH-6072 Sachseln

Phone +41 41 666 15 00  
Fax +41 41 666 16 50  
Web [www.maxonmotor.com](http://www.maxonmotor.com)

Otto Bock Healthcare Products GmbH  
Kaiserstrasse 39  
A-1070 Vienna

Phone +43 1 523 37 86  
Fax +43 1 523 22 64  
Web [www.ottobock.at](http://www.ottobock.at)